



# EndoGrip™

## Grasping Forceps

*Instructions for use.  
Read carefully prior to use.*

Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a physician.

### **Diversatek Healthcare**

#### **Corporate Headquarters**

102 E. Keefe Ave.

Milwaukee, WI 53212 USA

P 800.558.6408 | 414.265.7620

[sales@diversatekhc.com](mailto:sales@diversatekhc.com)

**[DiversatekHealthcare.com](http://DiversatekHealthcare.com)**

**Diversatek Healthcare EndoGrip™ Grasping Forceps**  
*Instructions for Use.*  
*Read carefully prior to use.*

**INTENDED USE**

The EndoGrip™ Grasping Forceps is used to remove tissue, stones, stents, or foreign bodies from the gastrointestinal tract.

<u>Part Number</u>	<u>Description</u>
1177-01	rat tooth alligator combo, 230 cm

**CONTRAINDICATIONS**

Including (but not limited to) coagulopathy.

**WARNINGS AND PRECAUTIONS**

1. Do not use if the package is damaged.
2. Do not use expired product.
3. Read the IFU carefully prior to use.
4. Prior to use, examine the device for any damage or malfunction such as bent components or inability to open and close the jaws. Do not use if damaged.
5. During use, gently advance the forceps to the retrieval site through the endoscope channel. If resistance is encountered during advancing, do not apply force to avoid damage to the endoscope.
6. This device is intended for single-use only. Do not re-use, reprocess, or re-sterilize single-use products. After use, dispose of according to local and institutional guidelines.
7. This product must be used by a trained physician.

**INSTRUCTIONS FOR USE**

1. Identify the object to be removed via endoscopic visualization and select suitable grasping forceps for endoscopy and the grasping target.
2. Ensure the grasping jaws are in closed position by pulling the handle toward the thumb loop and insert the grasping forceps into the biopsy channel of the endoscope.
3. Slowly advance the grasping forceps until it is visualized through the endoscope.
4. Advance the grasping forceps to the targeted object while maintaining visualization, pushing the slide handle to open forceps near the grasping target.
5. Tightly pull the handle to grasp the target, and withdraw the endoscope and grasping forceps together from the patient.

**POTENTIAL COMPLICATIONS**

Those associated with gastrointestinal endoscopy include, but are not limited to: swelling, bleeding, and infection.

*EndoGrip™ is a trademark of Diversatek Healthcare.*

*Diversatek™ is a trademark of Diversatek, Inc.*

**DESCRIPTION OF SYMBOLS USED ON LABELS**



Manufactured for



Use-by date  
(YYYY-MM-DD)



Lot Number



Part number

- |   |                                    |
|---|------------------------------------|
|  | Sterilized using ethylene oxide    |
|  | Temperature limit                  |
|  | Humidity limitation                |
|  | Atmospheric pressure limitation    |
|  | Do not use if package is damaged   |
|  | Keep dry                           |
|  | Keep away from sunlight            |
|  | Do not re-use                      |
|  | Consult instructions for use       |
|  | Not made with natural rubber latex |