

CleanFreak® Endoscope Cleaning Sponge with Enzymatic Cleaner

Instructions for use.
Read carefully prior to use.

INTENDED USE

The CleanFreak Endoscope Cleaning Sponge with Enzymatic Cleaner (1201-03) is used to safely remove debris from the outside surface of flexible endoscopes.

WARNINGS AND PRECAUTIONS

Never transfer a used cleaning sponge from one endoscope to another endoscope. Debris can be retained in the sponge and cause cross-contamination, which could result in patient complications. The cleaning sponges are single use only.

INSTRUCTIONS FOR USE

1. Remove sponge from packaging.
2. Fill a basin with 300 ml to 500 ml of water in order to properly dilute the detergent in the CleanFreak Dry Imbibed Sponge.
3. Squeeze and agitate sponge until a lather starts to form. The enzymatic cleaner is low foaming and does not require excessive foaming to function.
4. Use the sponge to wipe down the soiled endoscope. Rewet the sponge as needed.
5. Thoroughly rinse the endoscope.

PRODUCT DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of the product in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

DESCRIPTION OF SYMBOLS USED ON LABELS



Non-Sterile



Consult Instructions for Use



Do Not Re-Use



Not Made with Natural Rubber Latex



Use By Date (YYYY-MM-DD)

Diversatek Healthcare, Inc.
102 E. Keefe Ave.
Milwaukee, WI 53212 USA
P 800.558.6408 | 414.265.7620
sales@diversatekhc.com
DiversatekHealthcare.com

Made in the U.S.A.
*CleanFreak® is a registered trademark of Diversatek
Healthcare, Inc.*
Diversatek™ is a trademark of Diversatek, Inc.