

## CleanFreak® Endoscopy Pre-Cleaning Kit

*Instructions for use.*

*Read carefully prior to use.*

### **INTENDED USE**

The CleanFreak Endoscopy Pre-Cleaning Kit (1201-20) contains a wash basin and cleaning sponge with enzymatic cleaner to be used as part of the pre-cleaning process to safely remove debris from the outside surface of flexible endoscopes.

### **WARNINGS AND PRECAUTIONS**

**Never transfer** a used cleaning sponge from one endoscope to another endoscope. Debris can be retained in the sponge and cause cross-contamination, which could result in patient complications. The cleaning sponges are single use only.

### **INSTRUCTIONS FOR USE**

1. Use the wash basin as your endoscopy procedure room water basin. Fill the basin with 300ml to 500ml of water in order to properly dilute the detergent in the CleanFreak Dry Imbibed Sponge.
2. After the procedure, squeeze the sponge with enzymatic cleaner in the basin filled with water (this creates your enzymatic solution in the basin).
3. Wipe the scope insertion tube with the wet enzymatic sponge.
4. Suction the enzymatic solution in your basin through your scope.
5. Place the sponge over the tip of the scope (this protects the most vulnerable portion of the scope during transport).

### **PRODUCT DISPOSAL**

After use, this product may be a potential biohazard. Handle and dispose of the product in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

### **DESCRIPTION OF SYMBOLS USED ON LABELS**



Non-Sterile Device



Consult Instructions for Use



Do Not Re-Use



Not Made with Natural Rubber Latex



Use By Date (YYYY-MM-DD)

Diversatek Healthcare, Inc.  
102 E Keefe Ave.  
Milwaukee, WI 53212 USA  
P 800.558.6408 | 414.265.7620  
[sales@diversatekhc.com](mailto:sales@diversatekhc.com)  
[DiversatekHealthcare.com](http://DiversatekHealthcare.com)

Made in the U.S.A.  
*CleanFreak® is a registered trademark of Diversatek  
Healthcare, Inc.*  
*Diversatek™ is a trademark of Diversatek, Inc.*