

Diversatek Healthcare CleanFreak® Endoscope Transport Pad

Instructions for Use
Read carefully prior to use.

INTENDED USE:

The Transport Pads provide a safe and easy way to transport an endoscope to the reprocessing room after use.

INSTRUCTIONS FOR USE:

1. Remove pad from bag and place on cart.
2. Unfold pad to cover cart surface with cinch strings and absorbent section facing up.
3. After procedure, place coiled scope on center of pad.
4. Grasp strings and pull to cinch around scope for transport.
5. Dispose of properly.

DESCRIPTION OF SYMBOLS USED ON LABELS:



Non-Sterile Device



Do Not Re-Use



Do Not Use if Package is Damaged



Not Made with Natural Rubber Latex



Consult Instructions for Use



Date of Manufacture (YYYY-MM-DD)



Manufacturer



Lot Number



Part Number

Diversatek Healthcare, Inc.
102 E Keefe Ave.
Milwaukee, WI 53212 USA
P 800.558.6408 | 414.265.7620
sales@diversatekhc.com
DiversatekHealthcare.com

*CleanFreak® is a registered trademark of Diversatek
Healthcare, Inc.
Diversatek™ is a trademark of Diversatek, Inc.*