

# Capture™ 2-Tray Polyp Trap

*Instructions for Use.*

*Read carefully prior to use.*

## Diversatek Healthcare

102 E. Keefe Ave.

Milwaukee, WI 53212 USA

800.558.6408 | 414.265.7620

[sales@diversatekhc.com](mailto:sales@diversatekhc.com)

[DiversatekHealthcare.com](http://DiversatekHealthcare.com)

**Not made with natural rubber latex.**

**Does not contain DEHP.**

<u>Part Number</u>	<u>Description</u>
1202-02	Capture™ 2-Tray Polyp Trap, Non-Sterile

### INTENDED USE

The single use Capture™ 2-Tray Polyp Trap is used for suction retrieval of endoscopically removed polyps.

The single use Capture™ 2-Tray Polyp Trap consists of:

- One container with one filtered chamber to collect polyps, and one extra filtered chamber for additional usage.
- One port to the suction system, and the other port to the endoscope.
- Two Polyp Picks.

### WARNINGS AND PRECAUTIONS

- The Capture™ 2-Tray Polyp Trap is not designed to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- This single use medical device is not intended for reuse. Any institution, practitioner, or third party who reprocesses, refurbishes, remanufactures, resterilizes, and or reuses this disposable medical device must bear full responsibility for its safety and effectiveness.
- Two specimen retrieval strainers accompany the Capture™ 2-Tray Polyp Trap. Always insert the extra strainer into the Capture™ 2-Tray Polyp Trap upon removal of the first strainer; this practice will guard against inadvertent suctioning of a polyp through the Capture™ 2-Tray Polyp Trap and into the main suction system.

### DIRECTIONS FOR USE

1. Prior to clinical use, open the package and remove the device.
2. Familiarize yourself with the device(s) and read all the Instructions for Use.
3. Inspect the package and device for shipping or handling damage, i.e., cracked clear vessel, bent, misshapen, or missing specimen retrieval strainer(s), cracked, torn, or missing connection tube.  
If damage is evident **DO NOT USE THESE DEVICES, SAVE THEM FOR RETURN, AND CONTACT YOUR LOCAL PRODUCT SPECIALIST.**
4. Connect tube ① (see Figure 1) to the endoscope.
5. Connect port ② (see Figure 1) to the suction tube of suction system.  
**Caution:** Ensure the tubes do not kink. Kinked tubing will prevent the Capture™ 2-Tray Polyp Trap from functioning properly or may decrease or eliminate the ability to suction effectively.
6. Suction through the endoscope using standard endoscopic technique until the excised polyp is entrapped in the specimen retrieval strainer.
7. Remove the specimen retrieval strainer from the Capture™ 2-Tray Polyp Trap. Pull up the handle of the strainer to release (see Figure 2).

8. Insert the extra specimen retrieval strainer into the Capture™ 2-Tray Polyp Trap for additional usage. Close the strainer by pushing the handle down. Repeat steps 6 to 7.
9. The process of removing and replacing the specimen retrieval strainers in tandem (as needed), prevents loss of suction and allows the availability to retrieve additional polyps while the previously retrieved polyp is prepared by clinical staff.  
**Note:** To remove the retrieved polyp, use the provided Polyp Pick to transfer the polyp from the strainer to the appropriate specimen container for histopathology in accordance with institutional guidelines.
10. When the Capture™ 2-Tray Polyp Trap is no longer needed, disconnect the main suction tube from the Capture™ 2-Tray Polyp Trap, disconnect the Capture™ 2-Tray Polyp Trap tube ① from the endoscope, and reconnect the main suction tube to the endoscope. Close the strainer by pushing the handle down for disposal.

### PRODUCT DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of the product in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

### WARNING

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact Diversatek Healthcare to determine if additional information is available.

Report any serious incident that has occurred in relation to the device to Diversatek Healthcare.

Diversatek™ Healthcare is a trademark of Diversatek Inc.

Capture™ is a trademark of Diversatek Healthcare.

**DESCRIPTION OF SYMBOLS USED ON LABELS**

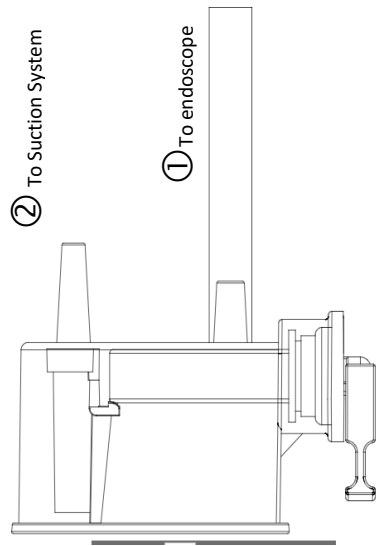









Figure 1

	Use-by date (YYYY-MM-DD)
	Lot number
	Manufactured for
	Part number
	Do not re-use
	Not made with natural rubber latex
	Consult instructions for use
	Keep dry
	Keep away from sunlight

**Rx Only (U.S.A.)** Federal law (U.S.A.) restricts this device to sale, distribution and use by or on the order of a physician.

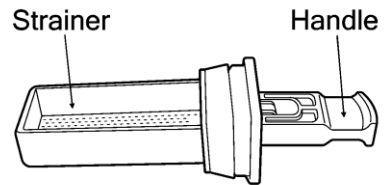


Figure 2