



Viper[®]

Dilation Syringe / Gauge

Instructions for Use
Read carefully prior to use.

Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a physician.

Diversatek Healthcare

102 E. Keefe Ave.
Milwaukee, WI 53212 USA
P 800.558.6408 | 414.265.7620

sales@diversatekhc.com

[DiversatekHealthcare.com](https://www.DiversatekHealthcare.com)

Diversatek Healthcare Viper® Dilation Syringe / Gauge
Instructions for Use
Read carefully prior to use.

INTENDED USE

The single-use Viper® Dilation Syringe / Gauge is used in combination with an inflation handle (not provided in this package) to inflate and deflate endoscopic balloon dilators while simultaneously monitoring balloon dilator pressures.

<u>Part Number</u>	<u>Description</u>
1204-01	Viper® Dilation Syringe / Gauge, Non-Sterile

INTENDED USER

Use of this device is restricted to a trained healthcare professional.

CONTRAINDICATIONS

Contraindications include those specific to dilation.

WARNINGS AND PRECAUTIONS

Syringe for single patient use only. Do not reuse, reprocess, or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing, or resterilization may also increase the risk of contamination of the device and/or cause patient infection or cross-infection, including but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

Do not use this device for any purpose other than stated intended use.

Store in a dry location, away from temperature extremes.

INSTRUCTIONS FOR USE

Assemble and use the Viper Dilation Syringe / Gauge according to inflation handle manufacturer's instructions.

This device is compatible with 60 cc syringe inflation handles.

PRODUCT DISPOSAL

After use, this product may be a potential biohazard. Dispose of product and packaging of the product in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

DESCRIPTION OF SYMBOLS USED ON LABELS

	Manufacturer
	Use-by date (YYYY-MM-DD)
	Lot number / Batch code
	Part number
	Non-sterile
	Do not use if package is damaged
	Do not re-use
	Do not re-sterilize
	Not made with natural rubber latex
	Consult instructions for use

Viper® is a registered trademark of Diversatek Healthcare.
Diversatek™ Healthcare is a trademark of Diversatek, Inc.