

DIVERSATEK HEALTHCARE IRRIGATION/BIPOLAR SET

For Original Malis Irrigator (#80-1164)
With Malis CMC-II or CMC-III Bipolar System

Part No. NU1060

Precautions:

Silicone has a low cut and tear resistance. Do not allow the silicone tubing portion of the set, between the plastic fittings, to come in contact with sharp objects.

Do not peel apart the tubing and electrical cord as this can cause the tubing portion to leak.

Do not use this bipolar cord set at generator power settings that exceed peak-to-peak RF voltage of 800 volts.



The material used for the manufacture of this device contains "DEHP" – Bis(2-ethylhexyl) phthalate.

Instructions for Use

1. Check your model of irrigation model. This device is only compatible with the Original Malis Irrigator #80-1164. **This device cannot be used with the Malis 1000 Irrigation Module.**
2. Shut the clamp off on the irrigation line and insert the spike into the irrigation container.
3. Insert the silicone tubing into the irrigation pinch valve assembly by pushing the LOAD button on the Malis Irrigator (#80-1164). To properly secure the silicone pinch valve tubing in the Irrigation Module, slide the plastic connectors through the slots provided.
4. Connect the male luer connector of the irrigation tubing set on the bipolar forceps.
5. Connect the two male plugs of the bipolar cord into the Bipolar System

IMPORTANT: Part No. NU1060 can only be used with the original Malis Irrigator (#80-1164). It is not compatible with the Malis 1000 Irrigation Module.

Set is supplied sterile and is intended for single use only. Do not use if the sterile package is damaged or the seal is broken in any way. Do not use any part that appears defective.

Diversatek Healthcare disclaims any and all responsibility and liability concerning the use of this product or any component herein, if it is reused, abused, misused, damages or used contrary to the Instructions contained herein.

*Malis is a trademark of Leonard I Malis, M.D. CMC-II & CMC-III is a registered trademark of Codman & Shurtleff, Inc.

REF Catalog Number



Caution (see accompanying documents)



Use By Date (YYYY-MM-DD)

LOT Lot Number



Sterilized Using Ethylene Oxide



Do Not Re-use

Manufacturer:

Diversatek
Healthcare

DIVERSATEK HEALTHCARE, INC.

102 East Keefe Avenue

Milwaukee, WI 53212 USA

P 800.558.6408 | 414.265.7620

sales@diversatekhc.com

DiversatekHealthcare.com

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.