

Your Procedure Appointment

Follow your doctor's instructions concerning your medications as listed below.

Medications	Instructions
_____	_____
_____	_____
_____	_____

Date of study:

Time of study:

Location:

Other instructions:

Diversatek
Healthcare

DiversatekHealthcare.com

P 800.558.6408 or 414.265.7620

Corporate Headquarters

102 East Keefe Avenue
Milwaukee, WI 53212 USA

orders@diversatekhc.com
sales@diversatekhc.com

Technical Research & Training Center

9150 Commerce Center Circle, Suite 500
Highlands Ranch, CO 80129 USA

technicalsupport@diversatekhc.com
clinicalsupport@diversatekhc.com
clinicaleducation@diversatekhc.com



Patient Information

MiVu™ Mucosal Integrity Testing

“My doctor says I need mucosal integrity testing. What should I expect?”

What is MiVu™?

MiVu™ Esophageal Mucosal Integrity Testing is a test to aid your physician in diagnosing specific esophageal conditions. MiVu is a minimally invasive testing system that is used during your endoscopy to measure the integrity or intactness of your esophageal mucosa.

Why is MiVu Being Recommended for Me?

MiVu is a safe, quick test performed during an endoscopy to check the health of the lining of your esophagus. It helps your doctor identify any damage or changes, which can assist in diagnosing conditions like GERD or EoE. This information is crucial for deciding the best treatment plan for you.



Understanding GERD and EoE

GERD (gastroesophageal reflux disease) happens when stomach acid or other contents flow back up into the esophagus, the tube that carries food from your mouth to your stomach. This can cause symptoms like heartburn, regurgitation, and chest pain.

EoE (eosinophilic esophagitis) is a condition when a type of white blood cell called eosinophils builds up in the lining of the esophagus. This can happen due to food allergies, acid reflux, or other triggers and can cause symptoms such as difficulty swallowing, chest pain, and food getting stuck in the esophagus.

What Happens During a MiVu Procedure?

MiVu testing is performed during your upper endoscopy (EGD), while under anesthesia. MiVu Endo Cap attaches to the distal end of the endoscope and is placed into your esophagus. There are special sensors on the device that gather measurements of your esophagus. Your physician will share the results with you when complete.

Preparing for the Test

There is no additional preparation for MiVu Mucosal Integrity Testing. Preparation for your upper endoscopy will be advised by your physician.



Potential complications are the same as those associated with gastrointestinal endoscopy.

Federal law restricts this device to sale by or on the order of a licensed healthcare professional.

Rx only.